Condicionador Ácido Fluorídrico 5% / 10%

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Hydrofluoric Acid Conditioner 5% / 10%

USE INSTRUCTIONS

INDICATION: The gel with 5% hydrofluoric acid is intended for conditioning the surface of porcelain and prosthetic parts for later cementation.

COMPOSITION: Hydrofluoridric acid, distilled water, thickening agent and pigment.

HOW TO USE: Confirm that the application tip is unobstructed. Isolate the outer surface of the denture to be conditioned. Apply 5% hydrofluoric acid to the inside of the prosthesis according to the type of material of the prosthetic part. After the conditioning time, wash thoroughly, removing all hydrofluoric acid and air dry. Follow the silane application and cementation protocol as directed by the product manufacturers.

WARNINGS: Professional use. Product ready to use. Follow the directions for

use. Use PPE to handle the product. Avoid contact with skin and mucous membranes. In case of contact with skin and/or mucous membranes, wash immediately with plenty of water.

PRECAUTIONS: Strongly acidic product. Avoid handling the product close to the patient. Discontinue use in case of allergic reaction. Keep out of the reach of children. Do not use the product after the expiration date. Do not use if the packaging is broken.

STORAGE: Keep the product in its packaging always closed. Protect from direct sunlight. Store the product at a temperature of 5 ° C to 28 ° C. Humidity up to 75%.

PRODUCT EXPIRY DATE: 24 months.

DISPOSAL: Follow current legislation for the disposal of dental products.

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