Arcos

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USE INSTRUCTIONS

FOLDABLE DAM FRAME:

Produced in sterilizable material, provided with claws for holding the rubber dam for absolute isolation of the teeth during clinical procedures.

INDICATIONS: Dental restoration and absolute isolation endodontic treatments rubber dam fixation.

USE INSTRUCTIONS: Fix the rubber dam in the Foldable Dam Frame; take all the set to patient's mouth, mark the tooth to be treated; drill the rubber dam in the marked location and fix it to the tooth using clamps.

OSTBY / U-SHAPED DAM FRAME:

Produced in sterilizable material, provided with claws for holding the rubber dam for absolute isolation of the teeth during clinical procedures.

INDICATIONS: Dental restoration and absolute isolation endodontic treatments rubber dam fixation.

USE INSTRUCTIONS: Fix the rubber dam in the OSTBY / U-SHAPED DAM FRAME; take all the set to patient's mouth, mark the tooth to be treated; drill the rubber dam in the marked location and fix it to the tooth using clamps.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: Should be done by autoclaving at a temperature of 121°C up to 135°C (249°F up to 275°F), observing the manufacturer's autoclaving instructions manual. Repeat the process for each use.

STORAGE: Keep the product in a designated and closed cabinet protected from dust and insects.

COMPOSITION: Polyacetal.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

SHELF LIFE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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